


Rigicon - Saline Filling Instruction

Appropriate size of testicular prosthesis will be determined for the individual patient by the physicians (*urologists, plastic and pediatric surgeons*).

The company does not supply needle for filling saline. During operation, the physician can use insulin needles (4-6 mm diameter) and injector (29-30-31G and 4-5mm lengths) for filling.

Fill the needle with the appropriate quantity of saline. Then, insert the needle into the silicone tab and empty all of the saline in the syringe into the testicular prosthesis.

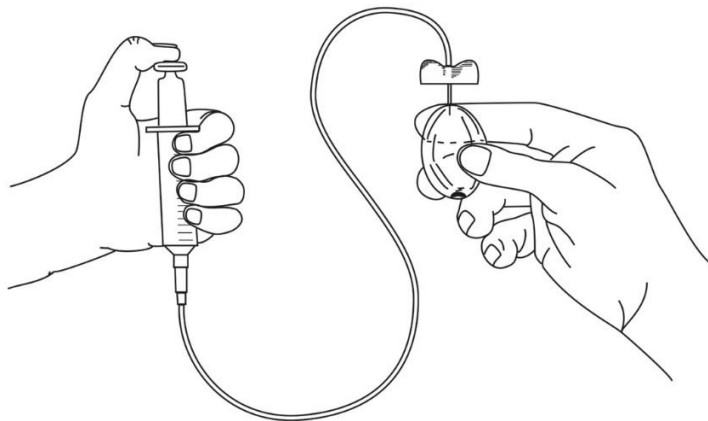

Each product comes with volume for filling the implant. 10% more or less saline filling is acceptable.

| Product Code | A (Length) | B (Diameter) | Volume |
|--------------|------------|--------------|--------|
| TestiSF-XS | 2.7 cm | 2.1 cm | 6 cc |
| TestiSF-S | 3.2 cm | 2.6 cm | 12 cc |
| TestiSF-M | 4.1 cm | 2.9 cm | 18 cc |
| TestiSF-L | 4.7 cm | 3.2 cm | 22 cc |
| TestiSF-XL | 5.1 cm | 3.2 cm | 26 cc |