

Rigicon[®]
Innovative Urological Solutions

ContiReflex[®]

Enhanced Artificial Urinary Sphincter



REDEFINING THE NEW GOLD STANDARD



■ Features and Benefits

Ideal AUS Envisioned by Experts¹

- » Easily manipulated and deactivated
- » Adaptable occlusive cuff pressure in a real-time manner
- » Simple and robust design
- » Implanted via a minimally invasive procedure

ContiReflex® The Innovative AUS

- » Pump easily manipulated, deactivated and activated
- » Smart PRB adapts occlusive cuff pressure in a real-time manner
- » Innovation combined with a simple and robust design
- » Implantation via a single-incision penoscrotal approach²

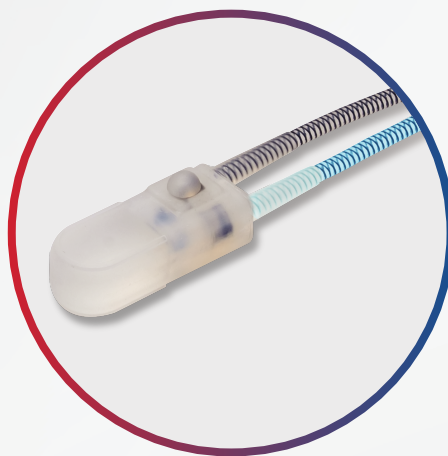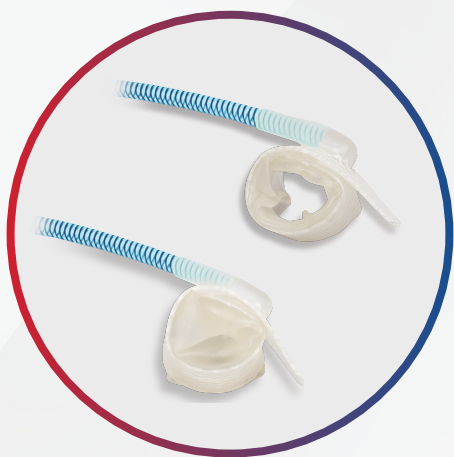


ContiReflex®

Enhanced Artificial Urinary Sphincter

The conventional AUS design does not modify occlusive cuff pressure in real-time. Cuff pressure is always limited by PRB's indicated pressure capacity.

The enhanced ContiReflex® smart "Reflex Balloon" senses IAP changes and modifies occlusive cuff pressure in a real-time manner. Cuff pressure is responsive to IAP changes and is no longer limited by the PRB's indicated pressure capacity.

Conventional AUS

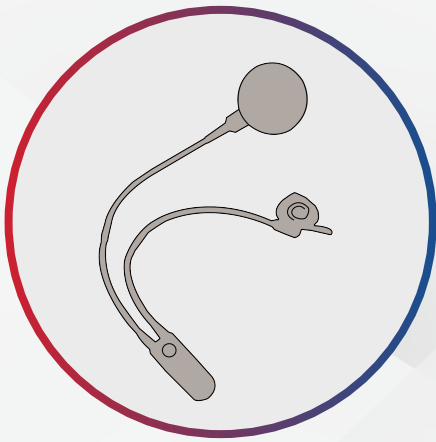

If IAP < 60 - 70 cm H₂O => cuff pressure = 60 - 70 cm H₂O

If IAP increases more than 60 - 70 cm H₂O




cuff pressure will remain still (60 - 70 cm H₂O)



No Instant Response

ContiReflex®

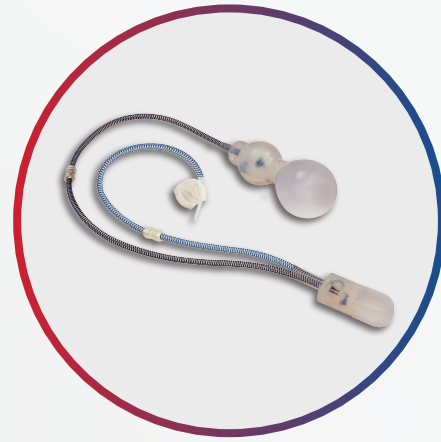

If IAP < 60 - 70 cm H₂O => cuff pressure = 60 - 70 cm H₂O

If IAP increases more than 60 - 70 cm H₂O



cuff pressure will match IAP



Real-time response with an innovative solution

PRB: Pressure Regulating Balloon
IAP: Intra-abdominal Pressure

ContiRegular® Occlusive Cuff

Product Code	Size (cm)
CTR-OC35	3.5
CTR-OC375	3.75
CTR-OC40	4.0
CTR-OC425	4.25
CTR-OC45	4.5
CTR-OC475	4.75
CTR-OC50	5.0
CTR-OC55	5.5
CTR-OC60	6.0
CTR-OC65	6.5
CTR-OC70	7.0
CTR-OC75	7.5
CTR-OC80	8.0
CTR-OC85	8.5
CTR-OC90	9.0
CTR-OC95	9.5
CTR-OC100	10.0
CTR-OC110	11.0
CTR-OC120	12.0
CTR-OC130	13.0

ContiReflex® Pressure Regulating Balloon

Product Code	cm H ₂ O
CTR-PRB49	40 - 49
CTR-PRB59	50 - 59
CTR-PRB69	60 - 69
CTR-PRB79	70 - 79
CTR-PRB89	80 - 89

ContiReflex® Control Pump

Product Code

CTR-CP

Conti® Accessory Kit

Product Code

CT-AK

▶ Scan the QR Code to see the **smart** Reflex Balloon and the occlusive cuff pressure change in action.



PRB: Pressure Regulating Balloon
IAP: Intra-abdominal Pressure

1 Biardeau X, Aharony S; AUS Consensus Group. et al. Artificial Urinary Sphincter: Report of the 2015 Consensus Conference. *Neurourol Urodyn.* 2016 Apr;35 Suppl 2:S5-7

2 Wilson SK, Westney OL, Mulcahy JJ. Twenty years later: is the scrotal one-incision AUS of value? *Int J Impot Res.* 2022 Apr;34(3):243-251. DOI: 10.1038/s41443-020-0317-6. Epub 2020 Jun 2. PMID: 32488211.

CAUTION: U.S. Federal law restricts these devices to sale by or on the order of a physician.

Indications, contraindications, warnings, and instructions for use can be found in the product labeling supplied with each device.

Please check availability with your local Rigicon sales representative or Rigicon Customer Service.

All images are the property of Rigicon.

All trademarks are the property of their respective owners.

Rigicon®
Innovative Urological Solutions

CNTR-04-01.2024

www.rigicon.com